


RapidXams™



Affordable multi drug screens @ (877) 747 TEST

ONE

Remove test from foil pouch.
Remove cap to reveal test strips.

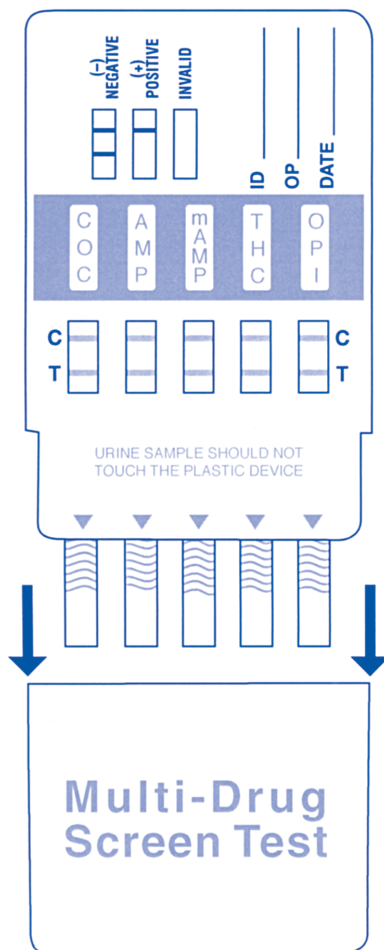

TWO

Immerse test strips in urine sample. Do not allow urine to touch plastic device.


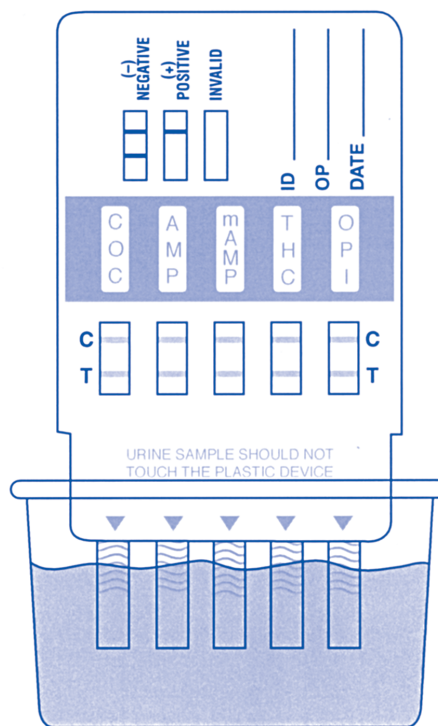


Illustration represents a 5 panel screen test. 1-12 panel configurations are available.

THREE

Read results. Results can appear within 1 minute. Tests can be photocopied for permanent results

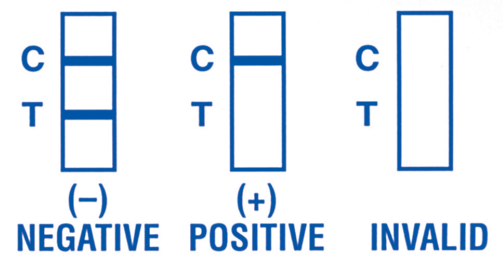

All positive results are presumptive and should be confirmed by an alternative method (e.g. GC/MS)

www.RapidExams.com

(877) 747-TEST

